

# Oroclean®

Cleaning Oil to dissolve earwax. Softens hardened earwax - the dirt and dead skin cells.

Medical device



#### Indication

For softening and dissolving of compacted earwax. Oroclean ear drops soften hardened earwax and the dirt and skin cells in it. Oroclean penetrates the wax and makes it water-soluble, which facilitates irrigation of the ear canal. Oroclean produces a local effect in the ear canal.

#### Directions for use

Warm the bottle between your hands, tilt your head slightly and instill 2–3 drops of Oroclean ear drops into the ear canal. Close the ear canal with a cotton ball to prevent the drops from running out of the ear. Let the ear drops work for about 10 minutes and then irrigate the ear with lukewarm water. If needed, a separate bulb syringe may be used for irrigation. Dry the ear with a towel carefully after irrigation so that no water is left in the ear. The procedure may be repeated, if needed.

#### Ear irrigation with bulb syringe:

Draw clean, lukewarm water into a bulb syringe. Place the tip of the bulb syringe carefully at the opening of the ear canal. Irrigate the ear by squeezing the bulb syringe gently. Repeat this until the water running out of the ear is clear. If you are uncertain how to use the syringe bulb, please consult a healthcare professional. For hygienic reasons, the preparation is meant for personal use only. Oroclean suits well even individuals with a hearing aid.  
*Note: Never insert a sharp-tipped object or a cleaning instrument into the ear canal.*

#### TECHNICAL INFORMATION

##### Composition

Almond oil, butylhydroxytoluene, butylhydroxyanisole, isopropyl isostearate, polysorbate 80, polyethylene glycol-40 sorbitol peroleate, tocopherol acetate, capric triglyceride and liquid paraffin.

##### Storage

Keep in room temperature (15–25 °C) to protect from sunlight. Keep out of the sight and reach of children. Do not use after the expiry date which is stated on the pack. The expiry date refers to the last day of that month. Any unused preparation may be taken to a pharmacy for disposal.

##### Pack type and pack size

White dropper bottle made of polyethylene plastic, plastic screw cap, package leaflet, and cardboard carton. 10 ml of ear drops in a 15 ml bottle.

#### Do not use Oroclean ear drops

- if you have ear pain or ear discharge
- if the eardrum has been punctured
- if you are allergic (hypersensitive) to a component in the preparation.

#### Warnings and special precautions for use

If you have ear pain or ear discharge, consult a physician because you could have an inflammation of the middle ear or some other condition requiring an examination by a physician. If you have repeatedly in the ear canal earwax that is hard to remove, you should be examined by a physician to find out whether the complaint is due to e.g. the structure of your ear canal.

#### Pregnancy and breast-feeding

This preparation can be used during pregnancy and breast-feeding.

#### Adverse effects

No known adverse effects. Local hypersensitivity reactions related to the skin of the ear canal may occur.